

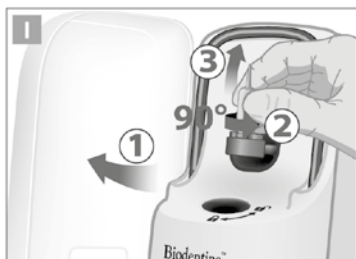
# Biodentine™ Mixer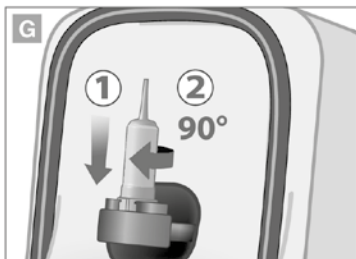

fr	Mixeur pour Cartouche	1
en	Cartridge Mixer	4
es	Mezclador de cartuchos	7
de	Patronen-Mixer	10
it	Miscelatore per cartuccia	13
pt	Misturador para Cartucho	16
nl	Patroonmixer	19
pl	Mieszadło do kapsulek	22
el	Συσκευή ανάμειξης φυσίγγων	26
bg	Смесител за патрони	29
cs	Zařízení pro míchání náplně	32
da	Patronblander	35
et	Kassetisegaja	38
hu	Patronkeverő	41
lt	Kapsulės maišytuvas	44
lv	Mikseris kasetnei	47
ro	Mixer pentru cartuș	50
sk	Zariadenie na miešanie náplne	53
sl	Mešalnik za kartušo	56
sv	Patronblandare	59
no	Patronblander	62

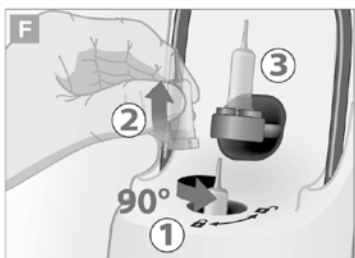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

## PRESENTATION

Box containing:

- 1 Main Unit
- 1 User Manual
- 1 to 3 Power Cords (according to the geographic area)

## PARTS AND ACCESSORIES

1. Lid
2. Lever
3. Mixing Arm
4. Activation Socket
5. Start/Stop button
6. Lid Magnet
7. Main Switch
8. Power Plug
9. Spare fuse
10. Power Cord

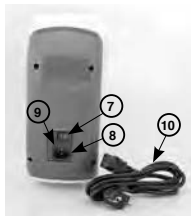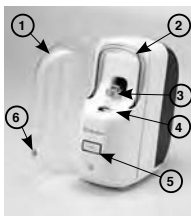(According to the geographic area)

## INTENDED USE

Activate and Mix the Biodentine™ XP cartridges.

## TARGET GROUP(S)

### Intended users

Dental healthcare professional only

## PERFORMANCE CHARACTERISTICS

### Technical performance / specifications

Activate and mix a cartridge containing powder and liquid

Dimension (Width x Depth x Height) : 240mm x 130mm x 270mm

Net Weight : 3.30kg

Voltage : accommodates from AC 100V to AC 240V 50/60HZ, 46W

Revolution : 6200rpm(max.)

Rated input : 100-240V, 50/60Hz

Power consumption : 46W

## INDICATIONS FOR USE

Biodentine™ Mixer is used to activate and mix dentine substitute from Biodentine™ XP cartridges for dental healthcare.

## CONTRA INDICATION

There are no contra indications

## EXPECTED LIFETIME

5 years

## INSTRUCTIONS FOR USE

### 1. Preparation

- 1.1. Wear the usual Personal Protective Equipment and No magnet
- 1.2. Choose the correct cable (if applicable) and plug in the power cord to the power socket
- 1.3. Switch the device ON (see picture A): the START button will light on (white color) with a beep sound.

### 2. Activation

The activation of the cartridge is done with the Mixer and not manually

- 2.1. Open the lid of Biodentine™ Mixer.
- 2.2. Put and align the Biodentine XP Cartridge in the activation socket (see picture B).
- 2.3. Lock the cartridge manually by making a **clockwise quarter turn** (see picture B).
- 2.4. Manually pull the lever (see picture C and D) down through all its course, (until it stops) a slight click may be heard.
- 2.5. Put the lever back to its initial place (see picture E)
- 2.6. Unlock the cartridge from the activation socket (see picture F step 1) by making a **counterclockwise quarter turn**.
- 2.7. Check if the cartridge is well activated : the cartridge piston has to be pushed inside.

**Warning** ; if not well activated, the cartridge may fall from the mixing arm.

- 2.8. Once the cartridge is activated, proceed to the Mixing (1 min30 max between activation and Mixing – Refer to Biodentine XP IFU) (see picture F step 3).

### 3. Mixing

- 3.1. Transfer the cartridge from the activation socket to the Mixing arm (see picture G step 1)
  - 3.2. lock the cartridge manually by making a **clockwise quarter** turn. (See picture G step 2)
  - 3.3. Close the lid (see picture H step 1)
  - 3.4. Press the Start button (see picture H step 2)
- Warning:** The START button is magnetically engaged, do not wear any magnet.
- 3.5. The START button will progressively become green indicating the mixing time elapsed (time of the mix : **30seconds**)
  - 3.6. At the end of the mix, the START button will blink green and will beep twice.
  - 3.7. Open the lid. (see picture I step 1)
  - 3.8. Unlock the cartridge from the mixing arm making a counterclockwise quarter turn and remove it. (see picture I step 2 and 3)
  - 3.9. Close the lid
  - 3.10. Use the cartridge following the Biodentine XP instructions for use.
  - 3.11. Turn off the mixer when further operation is no longer needed.

### WARNINGS

1. This device is designed to activate and mix **Biodentine™ XP 200 / Biodentine™ XP 500 cartridges** manufactured by Septodont.
2. Manipulate the device with appropriate personal protective equipment
3. Remove the foam around the mixing arm before first use
4. Always put Biodentine™ Mixer on a flat surface.
5. Please follow the instruction for use
6. For more information about the **Biodentine™ XP 200 / Biodentine™ XP 500**, please read the **Biodentine™ XP** cartridge instructions for use.
7. The lid must be closed before pressing the START button to allow the mixer to start running.
8. The mixing process starts by simply pushing the START button.
9. Speed level is electronically controlled and regulated to ensure a consistent speed.

### SIDE-EFFECTS

No known side effects.

Please report any serious incident that occurred in relation to the device to Septodont ([vigilance@septodont.com](mailto:vigilance@septodont.com)) and to the local competent authority for medical devices.

### TROUBLESHOOTING

Scenario	Cause	Solution
The device fails to function after pressing START button	Main switch has not been turned on	Turn on the main switch
	Power cord has come loose	Fully plug in the power cord
	Lid has not been shut	Make sure the lid is shut before mixing
The device moves while mixing	The device is not placed on flat surface	Make sure the device is placed on flat surface
The cartridge cannot be insert in the Mixing arm	The cartridge is not correctly activated	Place the cartridge on the activation socket and pull down the mixer lever until you hear the clicking sound
	The Mixer lever was not pulled down until the clicking sound	
The cartridge is ejected during the mixing	The cartridge is not correctly locked onto the Mixing arm	Open the lid, discard the cartridge.
The mix stops	The lid is opened during the mixing	Close the lid and press "START"
The START button blinks red with beeping sound (Failure mode)	The lid is opened for more than 10 seconds	Discard the cartridge.

The device has been tested to meet the following standards: IEC60601-1-2: 2014, EN60601-1-2: 2014, (EU) MDR 2017/745

### WARRANTY

SEPTODONT warrants that the product will be free from substantial defects in material and workmanship, and will substantially perform in accordance with the technical specifications set

forth in the description of the product for the duration of the warranty period. The disposable parts, such as the fuses are not included in the warranty.

All claims must be accompanied by a full description of discrepancy or malfunction. The warranty does not cover any misuse of the mixer.

Please visit <https://www.septodont/biodentine/warranty.com>

#### DISINFECTION AND CLEANING

For your own safety, please disconnect the device from the power supply before cleaning the unit.

Please use a normal wipe disinfection to keep the unit and power cord clean. Do NOT use solvent based or abrasive cleaning agents as they can damage the plastic components.

#### STORAGE

From -5°C to 40°C (23°F to 104°F) and protect from moisture

#### DISPOSAL

Don't mix with normal domestic waste

#### SYMBOL

	CE Mark
<b>Rx only</b>	For professional use only
	Manufacturer
	Date of manufacture
<b>EC REP</b>	Authorized Representative in the European Community
	Importer
	Distributor
	Consult Instruction for use
	Caution
	Type B applied part
<b>REF</b>	Catalogue number
<b>LOT</b>	Batch number
<b>SN</b>	Serial number
	Don't mix with normal domestic waste
<b>UDI</b>	Unique Device Identifier
	Fragile
<b>Power consumption</b>	Power consumption
<b>Rated input</b>	Rated input
	Temperature limit
	Keep dry
	Refer to the IFU before use
	Power symbol

	Lock position
	Unlock position

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**EC REP** Advena Ltd.
Motion Dental Equipment Corp.
SEPTODONT Inc